


ProOne Installation Instructions

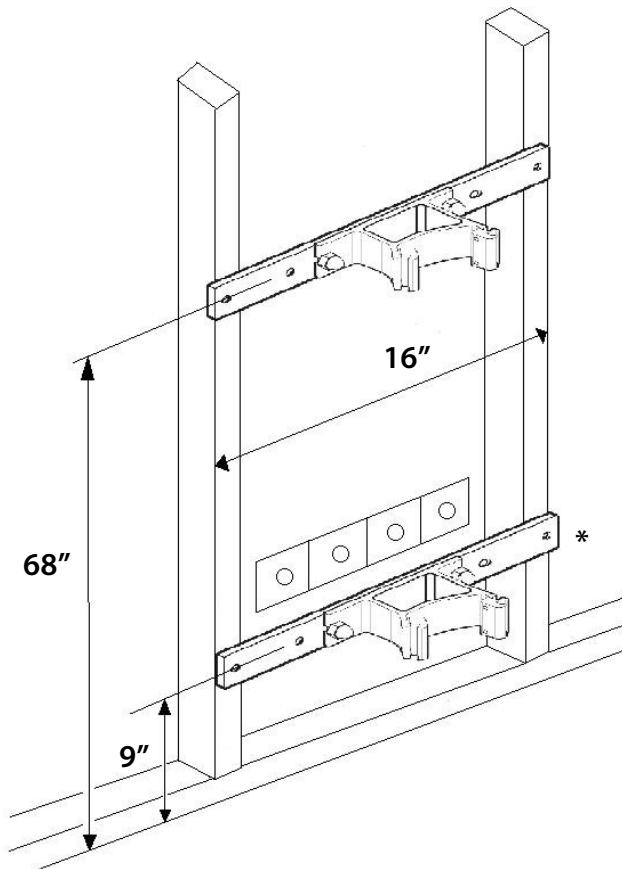

Wall preparation:

- The pan creates a pull-out force of 608 pounds.
- Wall bracket has 4 pre-drilled holes, 16" and 12" on center. Either one can be used.
- If metal studs are in wall, use a mounting board to expand the load over several studs to meet the pull-out requirement. If needed, sandwich the wall on both sides.

Mounting Options:

- Top wall mount (68 inches) and to the floor
- Two wall mounts (68 inches and 9 inches)*

*optional mounting configuration and the second mount must be ordered additionally. Part Number 10018380.

Input / Output Outlets

All outlets must be within 3 feet from the back of the column and 12-48" from the floor.

- Power: 110V, 20A dedicated circuit
- Remote exposure switch: 2"x4" electrical boxes at ProMax and remote exposure switch locations.
 - 1/2" conduit between 2"x4" electrical boxes required
 - All parts supplied for exposure switch connection
- Digital computer connection: LAN drop